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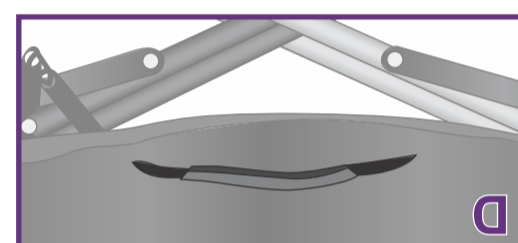
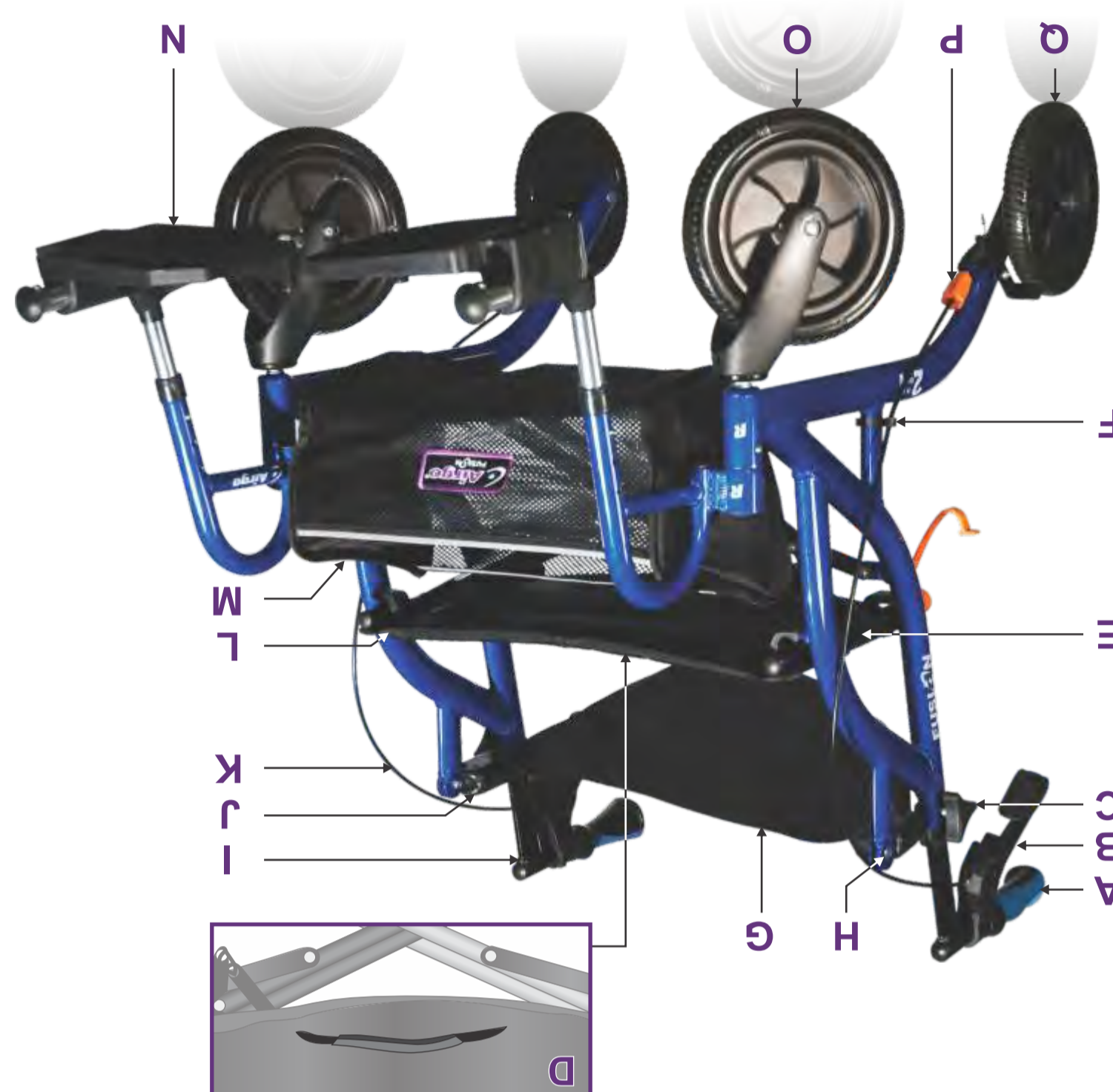
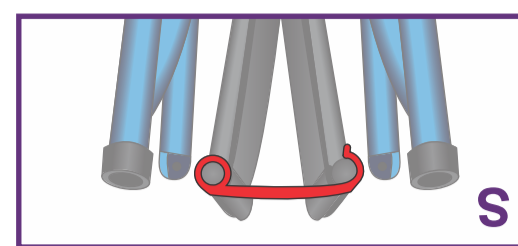
**S** Folding clip  
**R** Safety lock  
**D** Rear wheels  
**P** Brake adjustment knob  
**O** Front wheels  
**N** Footrests  
**M** Soft basket  
**L** Seat  
**K** Brake cables  
**J** Armrests  
**I** Handlebar  
**H** Backrest screw  
**G** Reversible backrest  
**F** Brake cable guide  
**E** Seat support  
**D** Folding strap  
**C** Hand screw  
**B** Brake handle  
**A** Handgrip

**EN LIMITED WARRANTY**

AMG Medical Inc. covers this Airgo mobility aid by a Lifetime warranty on the frame and a five (5) year warranty on brake handles against defects in materials and workmanship. All other parts and components are warranted for a period of 90 days. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part that has become defective.

For warranty service, you have two options:

1. Return your rollator to the Vendor where the unit was bought.
2. In Canada & the U.S. call AMG's Consumer Service at 1-800-363-2381 for a return authorization number.

Please ensure you provide us with the following information when you call: Date of purchase and serial number (on crossbar underneath the seat). Our Customer Service Team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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**EN CHECK ALL PARTS**

Carefully examine all parts for shipping damage. DO NOT use the Fusion if there is apparent damage or if any parts are missing. If there are missing or damaged parts refer to the warranty on the other side of this guide.

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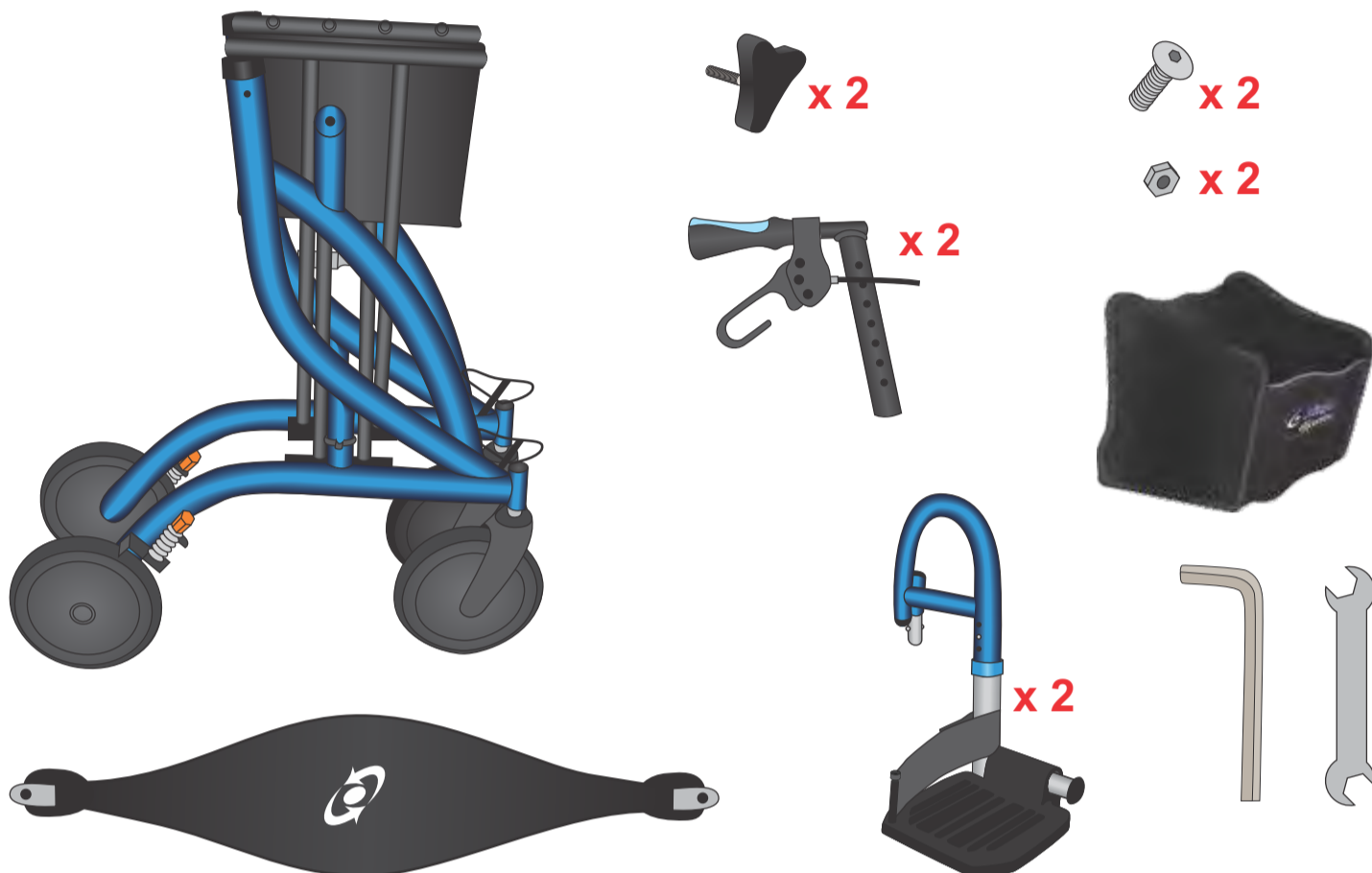
**EN WALKING TIPS**

1. Perform some simple stretches before walking. This will help you walk further and avoid injury.
2. Bring a bottle of water with you to stay hydrated.
3. Wear walking shoes that are well cushioned and snug in the heel, yet allow room to wiggle your toes.
4. When possible, walk on soft ground that is more joint-friendly such as grass or dirt paths. When walking near a road, wear bright clothing that is easily visible.

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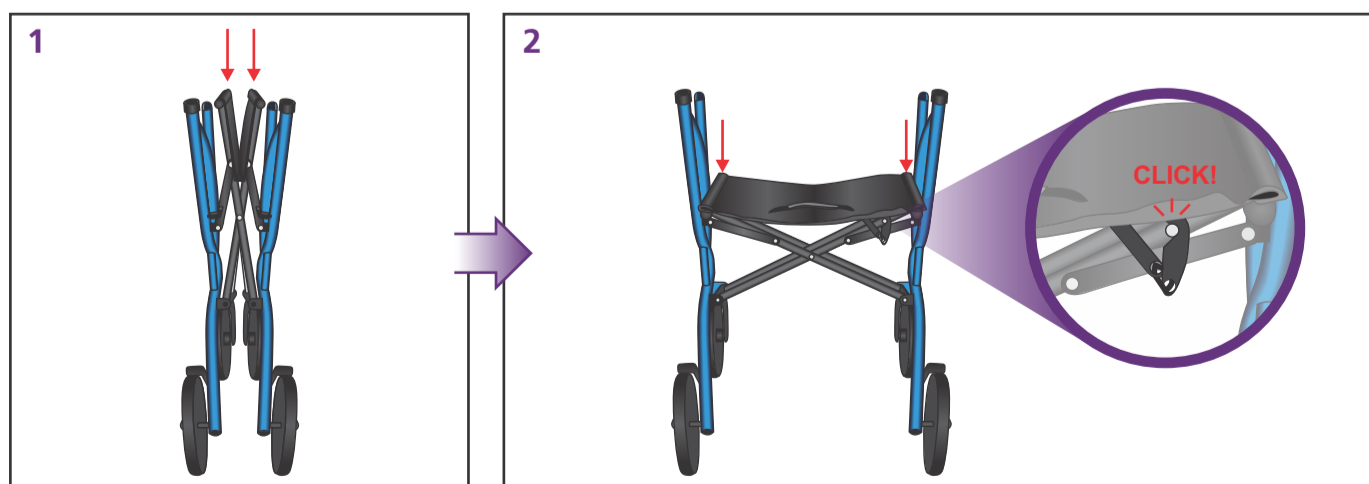
**EN UNFOLD THE ROLLATOR FRAME**

Place the frame standing upright on level ground. Push down on the Seat Supports (E) to expand and open the frame. You will hear the Safety Lock (R) click into place when the frame is fully unfolded.

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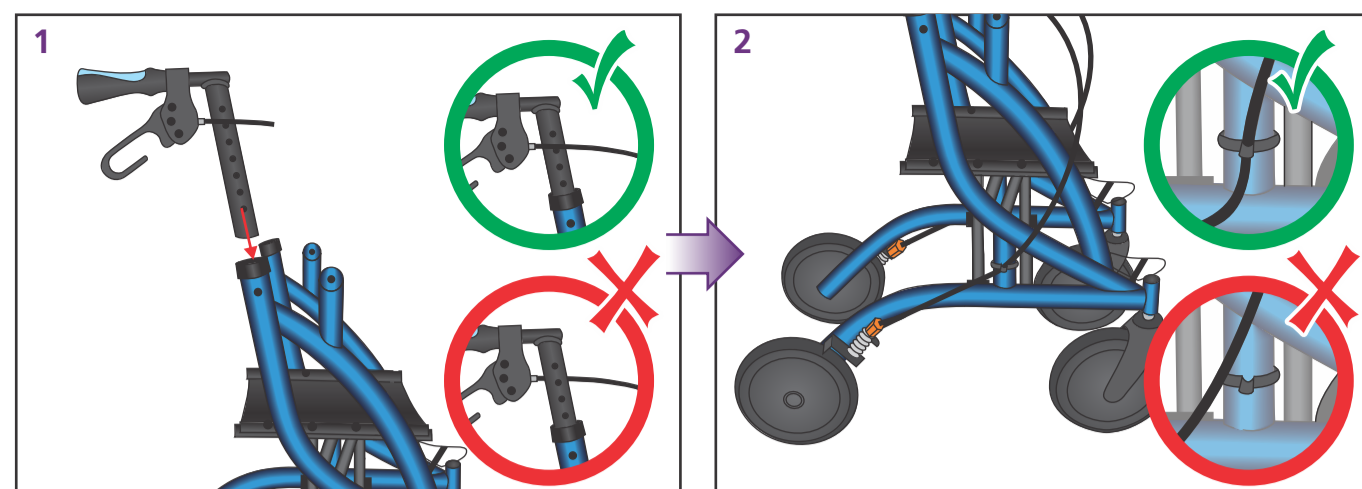
**EN INSERT THE HANDLEBARS**

Insert the Handlebars (I) into the openings of the frame, as shown. Ensure the Brake Cables (K) are on the outside of the handle and frame. Make sure the Brake Cable (K) passes through the Brake Cable Guide (F).

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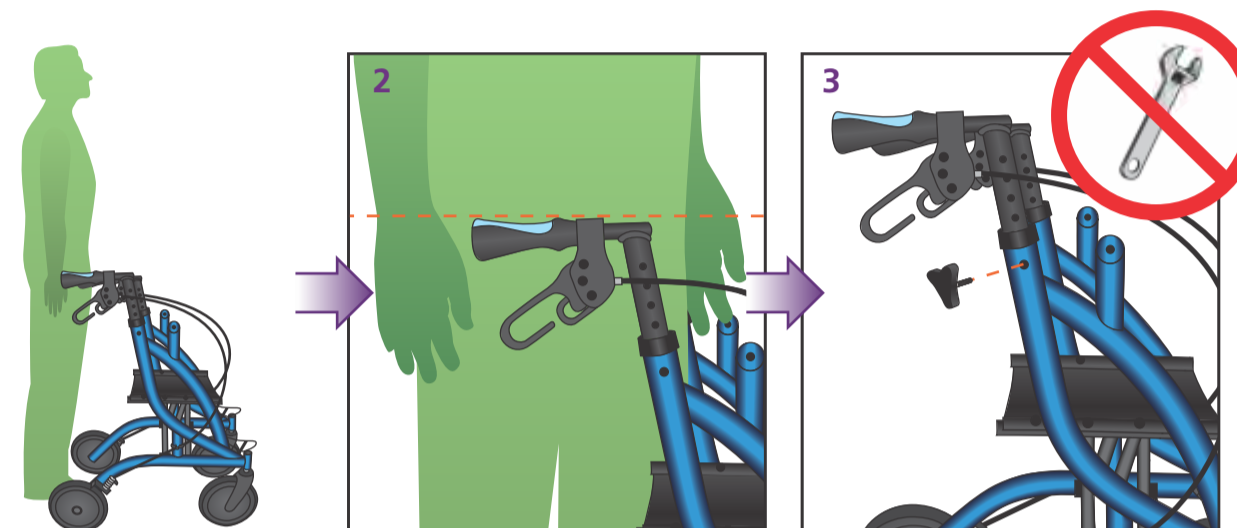
**EN ADJUST THE HANDLEBARS**

Stand upright behind your Airgo Fusion, shoulders relaxed, arms hanging at your side, and eyes looking straight ahead. In this position, adjust the Handgrips (A) to a height even with your wrists. Secure the Handlebars (I) at the closest hole to the desired height. Insert and tighten the triangular Hand Screws (C) until secure. HAND TIGHTEN ONLY.

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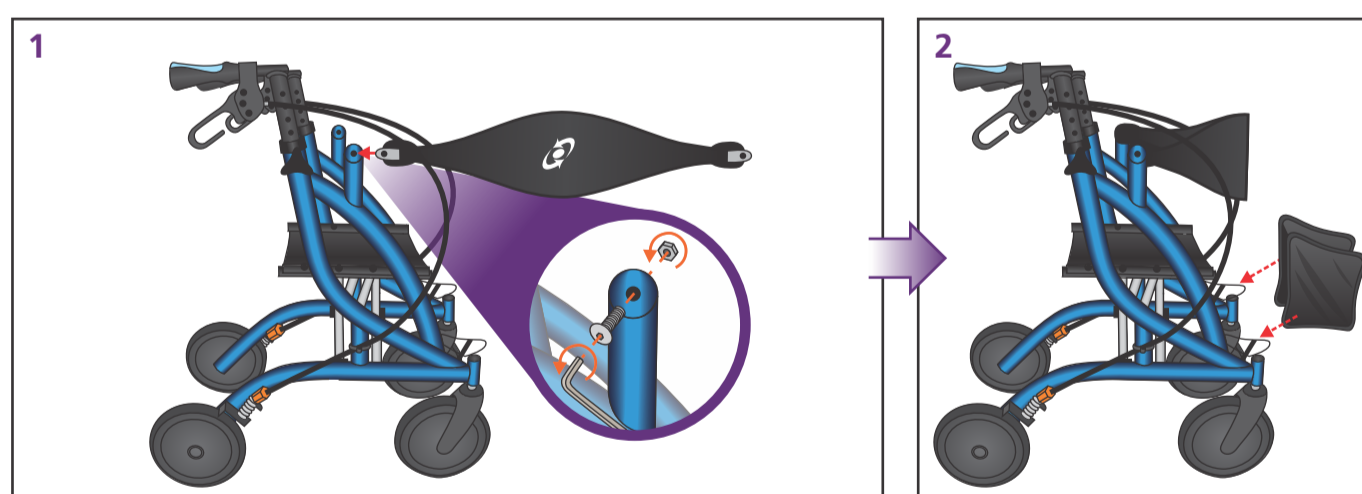
**EN INSTALL THE BACKREST & THE BASKET**

Insert the Backrest (G) so that the Fusion logo faces towards the Front Wheels (O). Slide one end of the Backrest into the fan bracket located near the Hand Screw (C). Slide the Backrest Screw (H) from outside of the unit through the hole in the fan bracket, then secure it with the nut and tighten using the Allen key and wrench provided. Repeat on the other side. Place the soft basket (M) onto the frame by slipping it onto the frame mounts. Start by slipping on by the back and then pulling toward the front.

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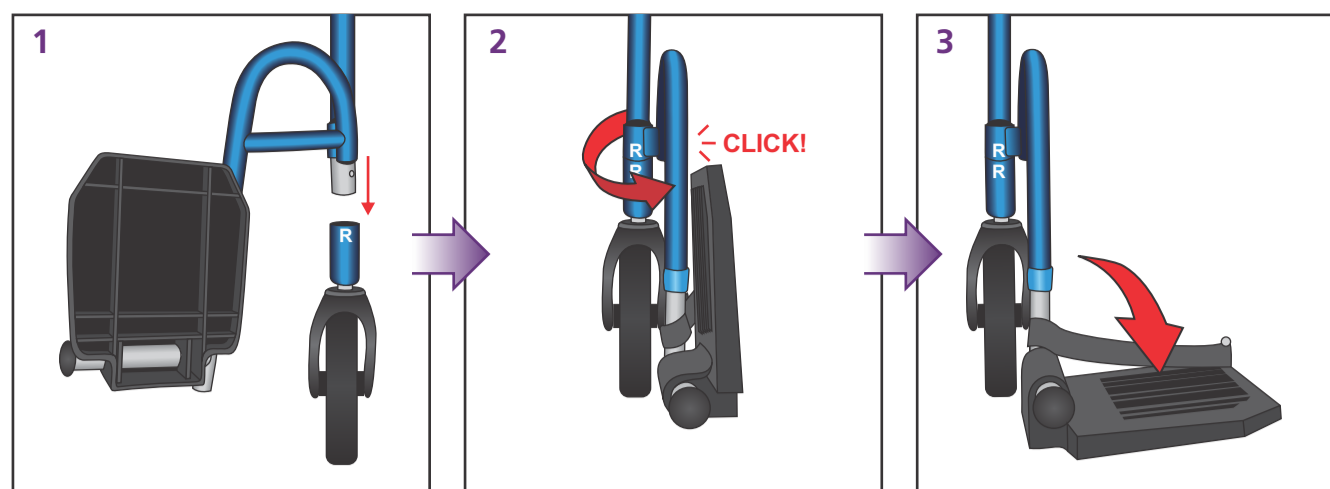
#### EN INSTALL THE FOOTRESTS

Match the letters on the frame (L and R) to the same letters on the footrests. Press the push-buttons on the footrest tubing and insert into the Fusion's frame. The footrests can be locked in two different positions (front, for transport chair use, and side, for rollator use). To rotate the footrests, press the push-button and rotate.

When not in use, the footrests can be conveniently stored by pushing the push-button and swinging them to the side of the frame where another push-button will hold them securely in place. This allows you to lift and store your Fusion safely. The footrests can also be removed when not in use. TIP: the black cap on the footrest tubing can be removed and used to cover the open tubing where the footrests slot into.

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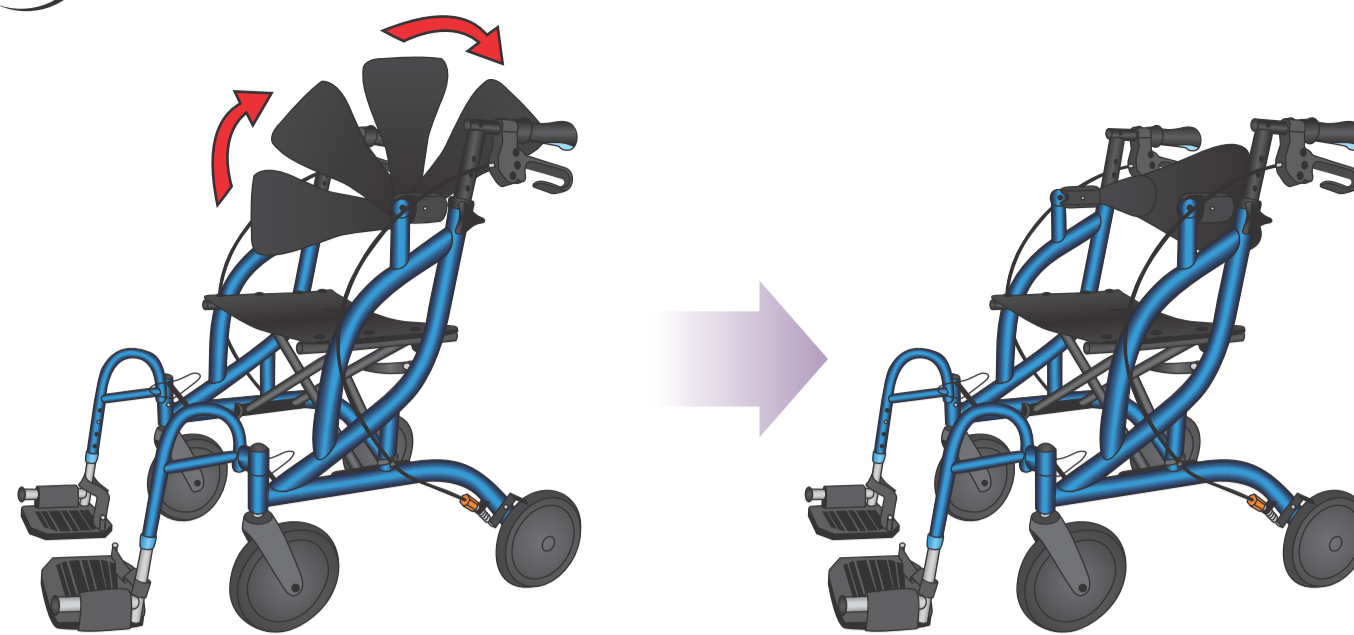
#### EN TRANSPORT CHAIR MODE

Flip the Backrest (G) toward the Handlebars (I) so that it is positioned above the front wheel and rotate the footplate toward the front of the Fusion until it locks into place. Before entering or exiting the Seat (L), always ensure that the footrest plates are folded up and that the brakes are locked. Once the user is seated, deploy the footrest plates as illustrated. Their height can be adjusted using the push-buttons on their tubing.

**CAUTION WHEN SEATED**  
Leaning out of the Fusion without proper assistance could cause tipping. To reduce the risk of an accident, NEVER reach or lean when you are seated in your Fusion. Instead, have someone present to help you.

**CAUTION WHEN PUSHING**  
When pushing someone in the Fusion, do not roll across small uneven surfaces. Doing so can cause the unit to tip forward. To safely cross curbs, uneven surfaces or use on large inclines, the person sitting on the Fusion must get out of the seat.

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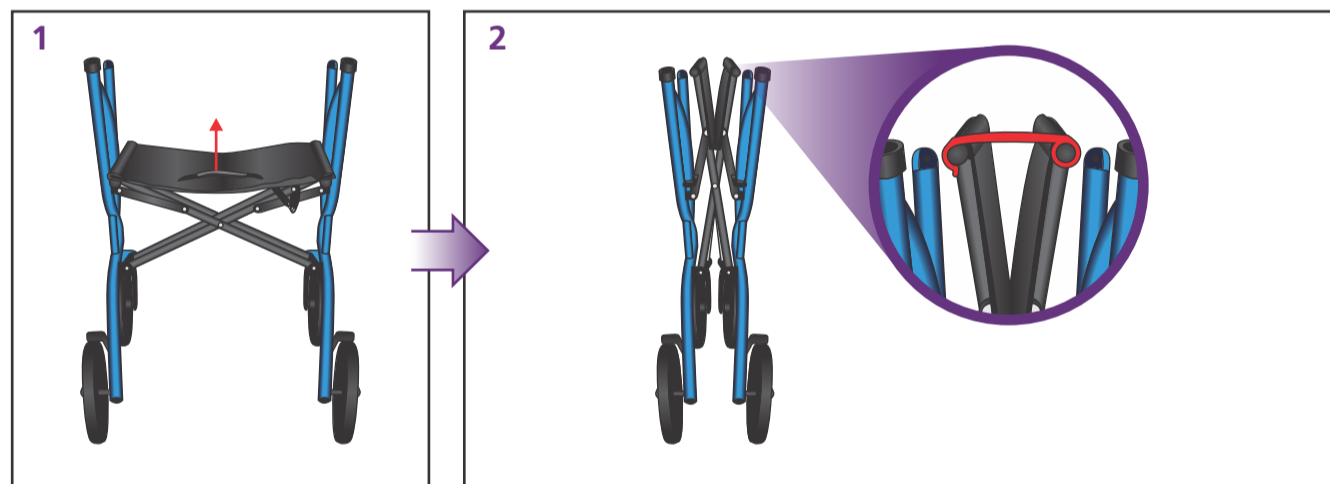
#### EN FOLDING YOUR FUSION

Before folding, lock the brakes by pushing down on the Brake Handles (B) until you feel or hear the brakes click into locked position. Pull the Folding Strap (D) located on the seat to release the safety lock and initiate folding. The position of the front wheels should be aligned to allow the unit to fold completely. Use the Folding Clip (S) to lock the unit in the folded position for safer lifting.

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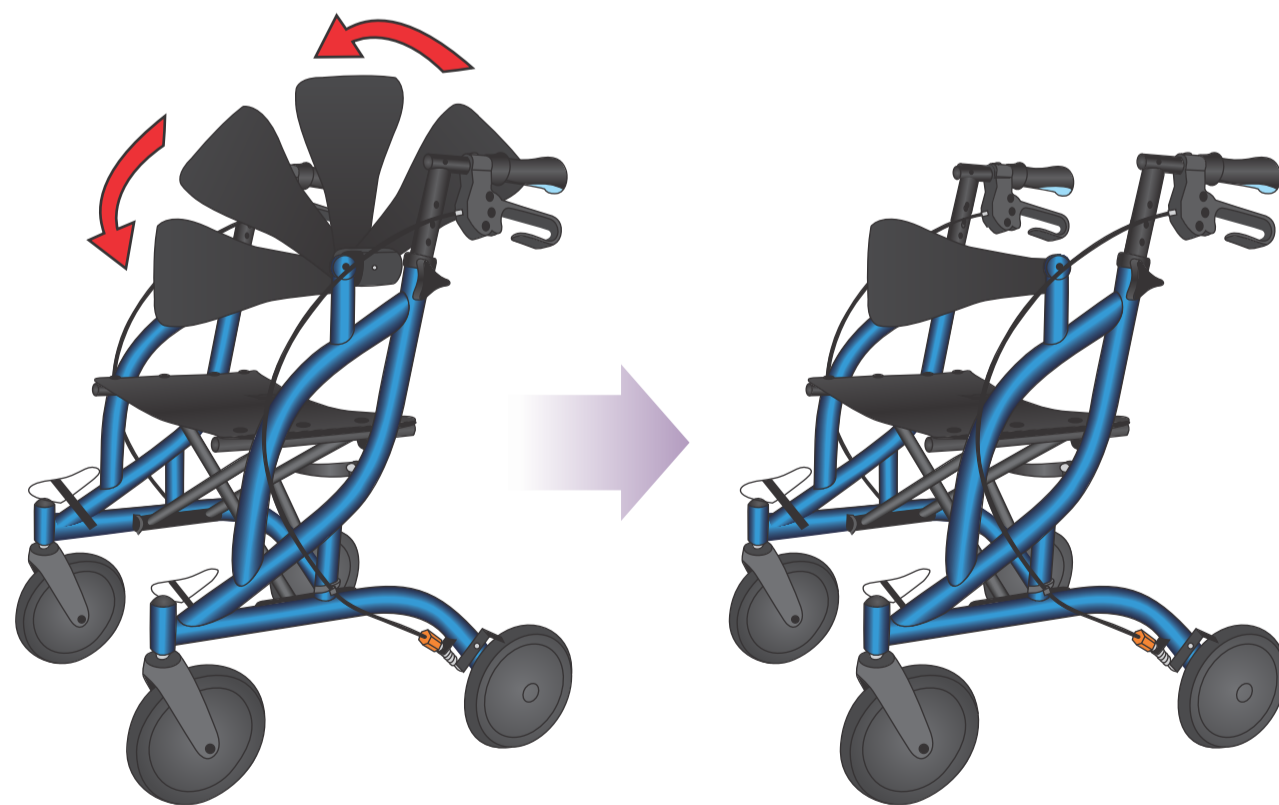
#### EN ROLLATOR MODE

Flip the Backrest (G) away from the Handlebars (I) so that it is positioned towards the front wheels. Before entering or exiting the seat always ensure the brakes are in the locked position by pushing down on both Brake Handles (B) until you hear them click.

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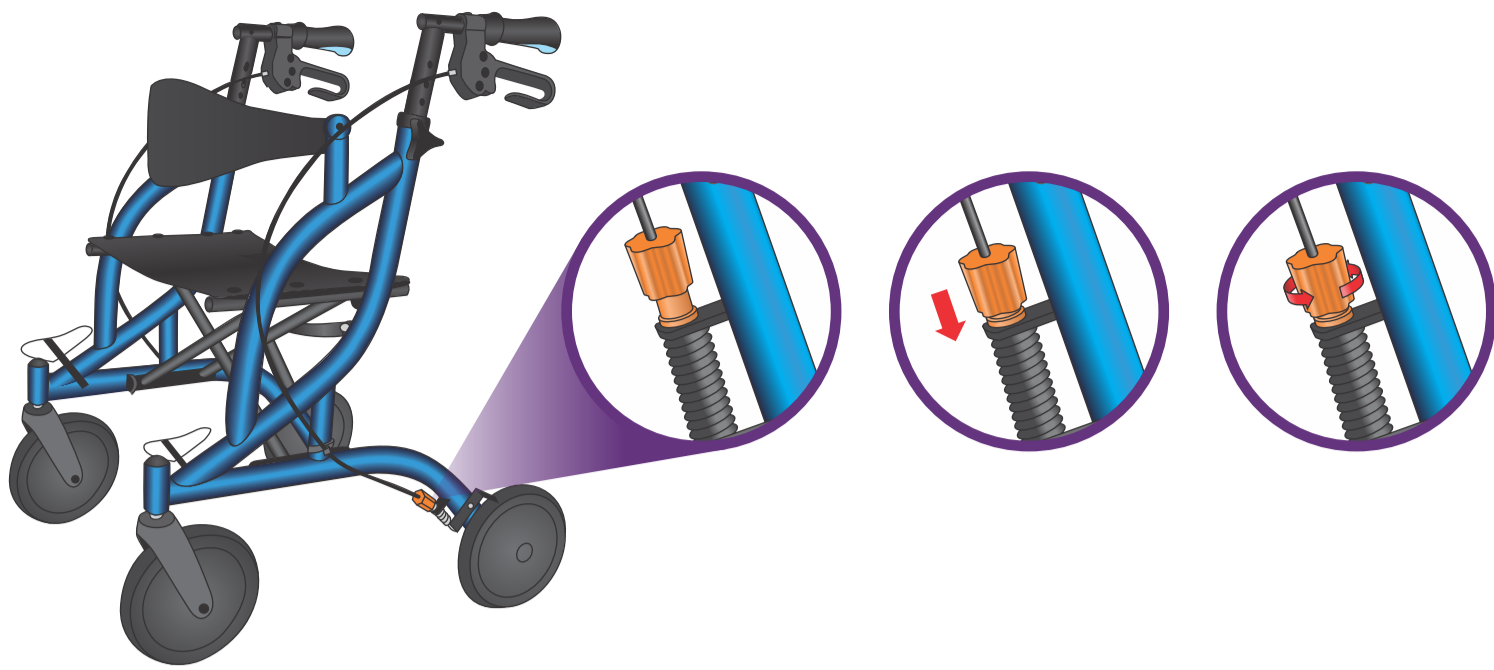
**EN PERIODIC BRAKE ADJUSTMENT**

To adjust the brakes, simply push down the orange Brake Adjustment Knob (P) and rotate. Rotating the adjustment knob clockwise will loosen the brakes while rotating it counter-clockwise will tighten them. **IMPORTANT** – Before using your Fusion again, ensure the brakes are working properly. **DO NOT** use it if it moves while the brakes are in the locked position. If you are unsure of how to adjust your brakes please see your authorized Airgo dealer or call our toll-free number (1-800-363-2381) for help.

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**EN READ THESE IMPORTANT SAFETY INSTRUCTIONS PRIOR TO OPERATING YOUR AIRGO® FUSION:**

- Consult your doctor, therapist or other qualified professional to determine if this Airgo® mobility aid will suit your particular needs.
- IMPORTANT** - Before using, make sure that all parts are secure and the frame is fully opened and in locked position.
- For proper positioning and use as a Rollator, imagine a line drawn between the back of the two Rear Wheels. Part of one foot should always be slightly ahead of that line as you walk.
- To slow down, squeeze the Brake Handles up. To lock the Brakes, push down on the Brake Handles until the Brakes click into locked position. If the Rear Wheels of the Fusion move while the Brakes are in locked position, **DO NOT** use the Fusion. Please see adjustment instructions in this manual or call our toll-free number: 1-800-363-2381(Canada & USA only) or see your distributor. To release the Brakes from locked position, pull up on the Brake Handles. Your Airgo® Fusion should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure.
- ALWAYS** lock the brakes before using the seat of your Airgo® Fusion. Risk of fall and serious injury may result if brakes are not locked while the user attempts to sit down. **DO NOT** navigate the Fusion independently while sitting on the seat, always have a caregiver present to help you. Falls are possible if the user leans or slides backward on the seat without first installing the backrest. Do not attempt to sit on the seat while on an incline and **NEVER** reach between your legs to pick up an object off the floor while seated. Doing so may cause the Airgo® Fusion to tip and a serious risk of injury may result.
- Get assistance if you are uncomfortable folding or unfolding your Airgo Fusion. **DO NOT** use your Fusion while it is folded. The stability of any mobility aid is compromised when the frame is in a folded position, increasing the likelihood of a fall and injury to the user.
- This product contains dry natural rubber.
- Do not exceed the following weight limit or serious injury could result:**  
Rollator Mode: 300 lb  
Transport Chair Mode: 300 lb

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